



CALIFORNIA
108 Whispering Pines Dr. STE 115
Scotts Valley, CA 95066
Toll-Free Phone: 800-293-3362
Toll-Free Fax: 866-808-7538
Phone: 831-430-9066
Fax: 831-430-9068

NORTH CAROLINA
500 Wait Ave Suite 1
Wake Forest, NC 27587
Toll-Free Phone: 800-606-7655
Phone: 919-556-8934
Fax: 919-556-0693

ONLINE
Email: contactus@reidsleeve.com
Web: www.lymphedema.com

Opera Instructions **(Initial Fitting must utilize enclosed stockinette)**

1. Pull the Opera up so it covers the length of the arm, fingertips may be slightly visible however most will cover the end of fingertips.
2. Pull the outer PowerSleeve over the entire length of the Opera, if using two PowerSleeves, pull the second PowerSleeve over the first one.

To Remove and Reapply

1. The outer PowerSleeve(s) must be removed first. The Opera may then be removed.

***If there is any discomfort while wearing your compression sleeve, remove sleeve and contact your health care provider.**

Cleaning Instructions

Washing

Hand wash in lukewarm water with a mild liquid dish washing soap. Do not use any soap that contains fabric softeners. Do NOT use Woolite! Do NOT dry clean!

Drying

For strapless garments squeeze out any excess water. For garments with straps, roll a towel up inside the sleeve and pull the straps as tight as possible to allow the towel to soak up any excess water. Now air dry the garment or place it in a dryer on the fluff cycle with **NO** or **LOW HEAT**. For garments with straps make sure that **all straps are closed** in the wearing position. For strapless garments Velcro close the garment in the wearing position. Place a tennis shoe in the dryer with the sleeve to keep it moving around for best results. These garments can take a long time to dry. For ReidSleeve Classic garments we recommend taking a full day to dry them in the dryer. **PowerSleeves included with garments should only be air dried.**

We offer a complete professional cleaning. Please contact us if you are interested in this service.